sedelogic

sits better | fits better

Sedelogic Lumbar Support car

NL – Handleiding

2.

DE – Anweisung

3.

UK – User manual

4.

Leverbare maten

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Art. 0711 26 cm hoogte, 37 cm breed.

Inleiding

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Art.0711 26 cm high, 37 cm wide.

Introduction

We thank you for choosing a Sedelagic Lumbar Support car. Only use the Sedelagic Lumbar Support after carefully reading this user manual.

The lumbar back cushion consists of 5 specific layers:
A cover which consists of a 3 mm soft 3D-woven polyester and polyamide fibre; Two 6.5 mm soft 3D-woven polyester fibre layers; A 5 mm hard layer consisting of a thermoplastic polyester fibre; and a 7.5 mm 3D-woven polyester fibre.

Installation

1. Remove the plastic protecting cover.
2. Place the lumbar support with the backside up.
3. Heat the material of the backside of the cushion with a hairdryer (make sure maintain a distance of 4 cm), until the hard core gets flexible. Caution: If the distance between hairdryer and cushion is too little (dependent on heat level of the hairdryer) the material can be damaged.
4. Sit down on a even chair and lean slightly backwards.
5. Place the lumbar support with the front side to the back of the user. The bottom of the cushion leans on the chair. Shape the cushion for 5 minutes; make sure it fits the back properly. The material will slowly harden.
6. Make sure to harden the cushion for 15 more minutes. If necessary iterate the above.
7. Your lumbar support is ready for use in your car.

Use

Check your skin frequently for any redness. In case of skin redness not disappearing within 15 minutes, discontinue the use and contact your therapist. When in doubt consult your therapist or dealer as well.

Flammability/ sharp objects

The material meets the need of present safety standards, nevertheless do not use or leave the lumbar support close to a naked flame or a high temperature heat source. Make sure to avoid contact with sharp edges.

Prevention

In case of a change in the ventilating and anatomic structure, the specific working mechanism cannot be guaranteed. Possible causes: High washing temperatures, not diluted chemical agents or sharp objects.

Cleaning and disinfection

Caution: do no steam or autoclave the support. Avoid contact with chemicals as well. The lumbar support cannot withstand temperatures above 60°C. These temperatures will change the materials structure.

The lumbar support can be disinfected with diluted antibacterial and antiseptical agents. Make sure to clean it with water before and after cleaning with this agent. Do not use a washing machine, dryer or fabric softener. Maximum washing temperature is 30°C.

Warranty

Innovetmed offers a warranty of 2 years after date of purchase against defects in materials and workmanship, provided that the product has been used properly. Remedies for breach of express warranties herein are limited to repair or replacement of the product. In no event shall damages for breach of any warranty include any consequential damages. Any damage arising from abnormal use is excluded from this warranty.

Complaints

For additional product information or complaints, please contact Innovetmed BV

Telephone: +31 45 2050966

E-mail: info@sedelagic.com

Website: www.sedelagic.com

Misprints reserved.
Patented product.