

## **Sedelogic Coolpad Manual**

1. Remove the plastic protecting cover.
2. Take out the cellulose insert.
3. Apply optionally gel on the cellulose insert.
4. Add the cellulose insert by the zip-fastener in the pad.
5. Secure the pad on the lower leg of the horse with the zip-fastener facing backwards.
6. Apply cold water with for instance a water hose.
7. The cellulose insert will swell, which compresses the area of the tendons and ligaments. This will either reduce swelling of this area or prevents it from occurring.
8. Remove the cool pad after maximally 2 hours.
9. The above can be repeated after 12 hours.

For more information: **[www.sedelogic.com](http://www.sedelogic.com)**